




User's Reference Card



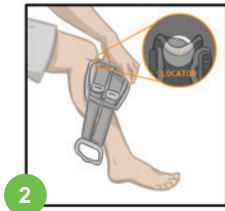

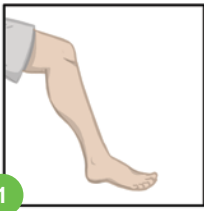


Putting On the L300 Go Functional Stimulation (FS) Cuff

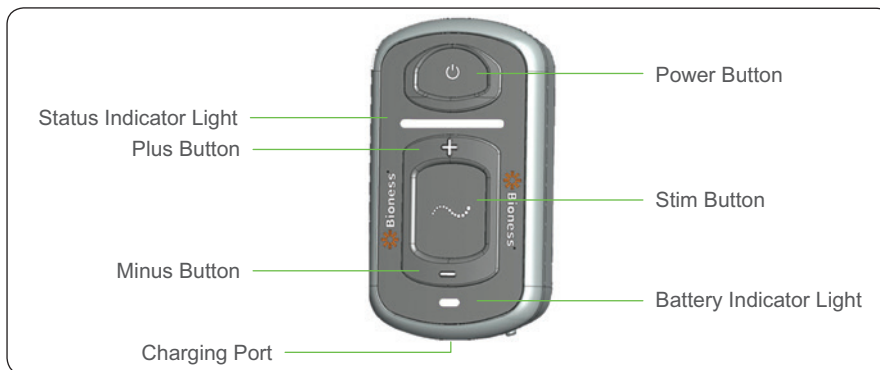- 1 While seated, slightly extend your lower leg.
- 2 Tilt the top of the L300 Go FS Cuff toward you, and then slide it up your leg until the locator rests below the kneecap. Lower the bottom of the L300 Go FS Cuff onto your leg.
- 3 Hold the L300 Go FS Cuff in place with your thumb or other hand on the EPG cradle. Pull the strap handle around the cradle.
- 4 Figure 4 shows the L300 Go FS Cuff correctly placed on the right leg.



Operating the L300 Go System



- 1 Press the  on/off button to turn on the EPG.
- 2 Select either gait or training mode:
 - To select gait mode, press the  mode button *briefly*.
 - To select training mode, press and *hold* the  mode button for 10 seconds. The EPG will acknowledge the new mode with audio and tactile feedback. Press the  mode button again to activate training mode. To exit training mode, press and hold the  mode button for 10 seconds.
- 3 If necessary, adjust the stimulation intensity level by pressing the  plus or  minus button.

EC REP

Manufactured by:
Bioness Inc
25103 Rye Canyon Loop
Valencia, CA 91355
Telephone: 800.211.9136
Email: info@bioness.com
Website: www.bioness.com

European Authorized Representative:
Bioness Europe B.V.
Stationsweg 41
3331 LR Zwijndrecht, The Netherlands
Telephone: +31.78.625.6088
Email: international@nl.bioness.com
Website: www.bioness.com




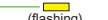


CE
0086

 **Bioness**[®]

Charging the Batteries






Connect the system charger set to the EPG, then plug into a wall socket.



 (flashing)	EPG Status Indicator Bar Flashes Red along with Battery Indicator Light Flashing Yellow	EPG Low Battery
 (flashing)	Flashes Yellow	Low Battery
 (flashing)	Flashes Green	Charging
 (flashing)	Solid Green	Fully Charged



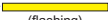




EPG Displays



 (flashing)	Flashes Green	System is On
 (flashing)	Flashes Green	Stimulation is Off
 (flashing)	Flashes Yellow	Stimulation is On
 (flashing)	Flashes Red with Audible Alarm	Active Error
	Solid Red	EPG Malfunction







Control Unit Displays*



	Solid	System is On
	Stim Wave Visible	System is in Gait/Training, Stimulation is enabled but Off
	Stim Wave Flashes	System is in Gait/Training, Stimulation is enabled and On
	Displays 0–9	Intensity Level
	Displays Gait Icon	Gait Mode is Enabled
	Displays Training Icon	Training Mode is Enabled
	Displays Audio Icon	Audio is On

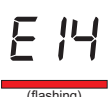*Optional Component

System Error Displays

 (flashing)	Faulty Electrode Contact. Ensure electrodes are snapped in, EPG fully in cradle. Clear error alert by pressing  or  on EPG.
 (flashing)	Low Battery Indicator on EPG. Charge system.
	EPG Malfunction. Press and hold  button for 30 seconds then power on or call Technical Support.

Rx Only (US only)